

How do you Create Perfection?

Why IPS Empress®?

With IPS Empress® you can offer your patients a clinically proven product, which provides excellent fit, wear, and the Ultimate In Esthetics.

- IPS Empress restorations wear like enamel.
- Over 25,000,000 IPS Empress restorations placed in the last 12 years.
- Indicated for Inlays, Onlays, Veneers and anterior Crowns.
- IPS Empress restorations are adhesively cemented.



IPSEmpress®

The ultimate in metal-free esthetics.

Call us toll free at 1-800-533-6825 in the U.S., 1-800-263-8182 in Canada. www.ivoclarvivadent.us.com
© 2002 Ivoclar Vivadent, Inc. IPS Empress is a registered trademark of Ivoclar Vivadent, Inc.

IPS Empress® Available From:

D&S Dental Laboratory, Inc.
1-800-236-3859
1-608-849-5343

100% CUSTOMER SATISFACTION
GUARANTEED!


ivoclar
vivadent
technical

IPS Empress® Preparation Guide

Anterior


Posterior

ANTERIOR CROWN PREPARATION

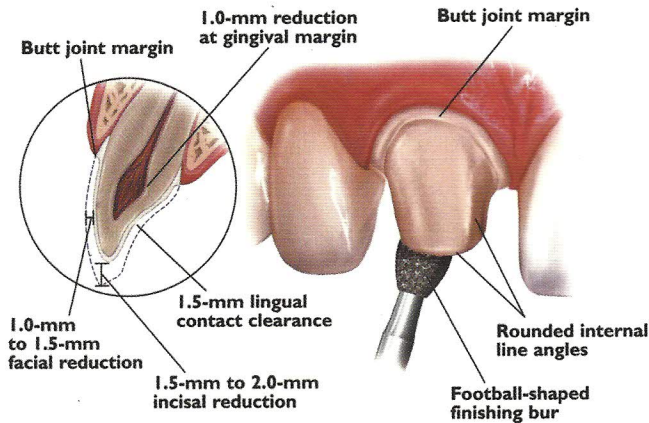


UNIFORM FACIAL PREPARATION

A medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

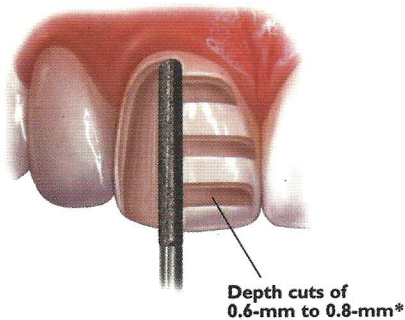


INCISAL PREPARATION

The diamond bur is angled to bevel back the incisal edge.

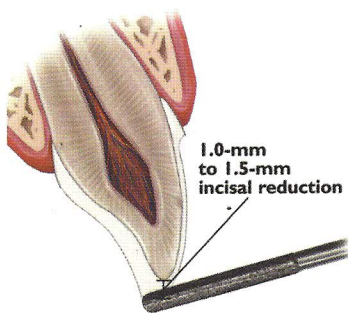


CHAMFER MARGINS


Correct preparation of the chamfer margins interproximally allows the appropriate bulk of porcelain.

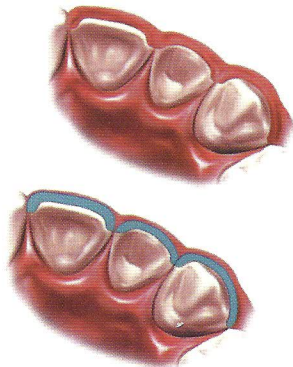

POSTERIOR CROWN PREPARATION

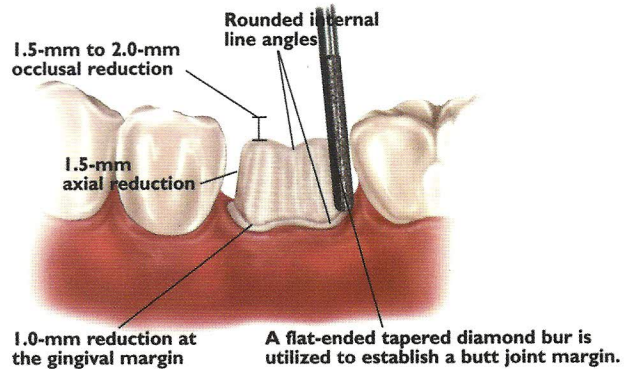

INLAY PREPARATION

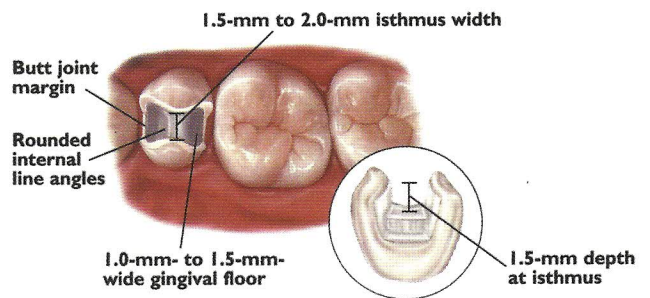

ONLAY PREPARATION

